# PRODUCT MANUAL

# Medistrom Pilot Flex

## Battery and Backup Power Supply

Thank you for purchasing the Medistrom  
CPAP Battery Portable Power Supply.

Please read this product manual carefully  
before putting the equipment into operation  
for the first time.

# Contents

## English

1. PILOT FLEX	3
1.1 PACKAGE CONTENTS	3
1.2 PAP DEVICE COMPATIBILITY	3
1.3 IMPORTANT SAFETY USAGE INFORMATION	4
1.4 INTENDED USE AND COMPATIBILITY	5
1.5 ABOUT YOUR CONTROL PANEL	5
1.6 SETUP	6
1.7 SPECIFICATIONS	8
2. CARE INSTRUCTIONS FOR PILOT FLEX	8
3. TRAVELLING WITH PILOT FLEX BATTERY BACKUP POWER SUPPLY	8
4. STORAGE AND DISPOSAL OF YOUR PILOT FLEX BATTERY BACKUP POWER SUPPLY	9
5. WARNINGS AND SAFETY PRECAUTIONS	9
6. WARRANTY INFORMATION	10
7. TROUBLESHOOTING GUIDE	11

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THIS PACKAGE INCLUDES:

1x Pilot Flex

1x DC Output Cable  
for ResMed AirMini

1x Short Charging Adapter  
Cable for ResMed AirMini

### ADDITIONAL ACCESSORIES FOR PILOT FLEX (SOLD SEPARATELY)

1. Cable Kit for AirSense 11
2. Cable Kit for AirSense 10 / 3B Medical Luna G3
3. Cable Kit for Resironics S60 / DreamStation 1 / 2
4. Cable Kit for DreamStation Go / BMC Medical Luna
5. Cable Kit for Lowenstein Prisma Smart
6. Cable Kit for Apex Medical iCH / Smart Med Ibreeze / Sefam S-Box
7. Medistrom AC power adapter
8. Medistrom 50W Solar Panel
9. Car Charger for Pilot-Flex, 12 Lite & 24 Lite

## 1.2 PILOT FLEX PAP DEVICE COMPATIBILITY

### SUPPORTED PAP DEVICES

The Pilot Flex can ONLY be used with the following APAP/CPAP devices:

- ResMed AirSense 11 Series (cables sold separately)
- ResMed AirSense 10 series (cables sold separately)
- ResMed AirMini
- Phillips Respironics DreamStation Go series (cables sold separately)
- BMC Medical Luna (cables sold separately)
- Apex iCH series (cables sold separately)
- DreamStation 1 and 2 Series (cables sold separately)
- Lowenstein Prisma Smart (cables sold separately)

Pilot Flex is designed to be charged using the original ResMed AirSense 10 series AC power adapter directly.

It can also be charged using the AirSense 11 series power adapter by using the short charging adapter cable for the ResMed AirSense 11.

It can also be charged using the AirMini series power adapter by using the short charging adapter cable for ResMed AirMini. Charge time via the ResMed AirMini AC adapter is approximately 7-9 hours.

For faster charging times, please use Medistrom AC Power adapter.

Pilot Flex can be also charged with the Apex iCH via the Apex iCH Short Charging Adapter Cable (sold separately), and the Medistrom Pilot Lite AC adapter (sold separately).

For more detailed instructions on charging, please see SECTION 1.6 Pilot Flex Setup, Step 1.

**NOTE:** For complete compatibility information including compatibility for Bi-Level devices, please visit [medistrom.com](http://medistrom.com) or contact Medistrom technical support at 1-877-407-4791 or email [support@medistrom.com](mailto:support@medistrom.com). Medistrom is not responsible for any damage caused by the improper use of the Pilot Flex.

### **1.3 IMPORTANT PILOT FLEX SAFETY USAGE INFORMATION**

Using a humidifier or heated tubing while powering your compatible PAP device with the Pilot Flex will significantly decrease the battery run time. For optimal run times please disable and/or detach any humidifiers and heated tubing.

- DO NOT expose battery to temperatures above 40°C / 104°F (Fire Hazard!)
- DO NOT expose battery to temperatures below -10°C / 14°F (Cell Damage Hazard!)
- DO NOT expose battery to relative humidity above 85% (Electrical Hazard!)
- Normal operating ambient temperatures are 0°C / 32°F to 35°C / 95°F
- Fully charge battery before storage and at least once every 6 months thereafter

## 1.4 PILOT FLEX INTENDED USE AND COMPATIBILITY

Pilot Flex is manufactured with genuine LG/Samsung industrial grade rechargeable lithium-ion cells. The Pilot Flex can be used as a backup power source to prevent interruption to PAP therapy in the event of a power outage.

Pilot Flex can serve as a portable PAP battery during travel by air, land, or sea when the main power source is not available.

Pilot Flex is designed to be used with most 24V and 12V PAP Devices (Refer to Section 1.2 PAP Device Compatibility)

Pilot Flex has a USB port (output: 5V 2.4A) which can be used to charge cellphones, tablets and other commonly used handheld devices.

## 1.5 ABOUT YOUR PILOT FLEX CONTROL PANEL

**1. Charge Port** Plug the compatible AC adapter into this port (use the short charging adapter cable if required). Ensure that the AC adapter is plugged into the main power outlet. Once the battery is charging the 4 green Charge Level Indicator Lights will turn on.

**2. Multi Function Button** Pressing the button will show the battery's charge level (if the battery is in Standby Mode: refer to Stand-by Mode in Section 1.6).

**3. Voltage "ON" Indicator Light** A light will shine beside the 12V or 24V indicating the current output voltage of the battery. If the battery malfunctions, the lights will output errors codes. Refer to Section 7 for a guide to errors codes.

**4. Green Charge Level Indicator Lights** When the battery is being charged, the lights will blink, and then shine solid green showing the level of charge going from 1 light to 4 lights. Refer to Section 7 for a guide to charge levels. Press the Multi Function Button to turn on the battery.

**5. USB-A Port (5V/2.4A, 9V/2A, 12V/1.5A)** Can be used to charge handheld devices (please check that the output Voltage and Amperage is compatible with the device that needs charging via the USB before connecting). Once connected press the Multi Function Button to turn on the battery.

**6. USB-C Port (Output: 5V/3A,9V/3A,12V/3A,20V/2.25A and Input: 5V/3A,9V/3A,12V/3A,20V/2.25A)** The USB C port can be used to charge portable devices in the output capacity and will supply a maximum of 45W. The Pilot Flex can be charged using the USB C port with a maximum input of 45W. Please note that the USB C port cannot be used while a PAP device is connected to the battery.

**7. DC Out Port** This port is used to connect a compatible PAP device to the battery.

## 1.6 PILOT FLEX SETUP

**ATTENTION:** Power adapters are NOT included with your Pilot Flex Battery and Backup Power Supply. Please refer to section 1.2 PAP Device Compatibility on information how to charge your battery.

### Using Pilot Flex as a Backup Power Supply for PAP devices

#### STEP 1 – Connecting Pilot Flex to AC Power Adapter

Plug the original ResMed AirSense 10 AC power adapter into the Charge Port of the Pilot Flex and plug the other end into the power outlet.

**NOTE:** Please use the Short Charging Adapter Cable for ResMed AirMini (included with Pilot Flex) to plug the original ResMed AirMini AC power adapter into the Charge Port of the Pilot Flex.

#### STEP 2 – Connecting Pilot Flex to PAP Device

Plug the L-shaped yellow tipped end of the DC Output Cable for ResMed AirSense 10 Series into the DC Out Port of your Pilot Flex and plug the other black tipped circular end into the power connector port of your AirSense 10 PAP device.

Once the Pilot Flex is connected to your PAP device, the Charge Level Indicator Lights will turn on briefly and then the Voltage Indicator Light will turn solid white showing the output Voltage. At this point your PAP device will show that it is receiving power.

If your PAP device is not receiving power, please check the cable connections and press the Multi Function Button.

### **Stand-by Mode**

If the PAP device has been connected but not used within 10 min (or the PAP device is turned off for 10 min), the Pilot Flex will go into Stand-by Mode to conserve power. To wake the Pilot Flex press the Multi Function Button.

### **Charging Pilot Flex**

Follow STEP 1 above. Charge times for the Pilot Flex via the ResMed AirSense 11, 10, Philips DreamStation 1 and 2 or the Medistrom Pilot Lite AC adapter is approximately 2-3 hours. Charge time via the ResMed AirMini AC adapter is approximately 7-9 hours.

**NOTE:** The ResMed AirMini AC adapter can only be used to charge the Pilot Flex and cannot be used in the Backup Power Supply configuration.

To use the ResMed AirMini PAP Device in the backup configuration with Pilot Flex please use the Medistrom AC Power adapter for Pilot-12/24 Lite (Sold separately), or the original ResMed AirSense 11 AC power adapter, the original ResMed AirSense 10 AC power adapter, the original Philips DreamStation 1 AC power adapter, the original Philips DreamStation 2 AC power adapter.

The Pilot Flex can also be charged via the USB C port with a max input of 45W. (USB C cable and USB C wall charger not included). Charge time via the USB C cable and 45W wall charger is approximately 4-5 hours.

**NOTE:** The Pilot Flex cannot be used in backup configuration with a USB C cable and wall charge.

### **Using Pilot Flex as a Portable Battery for compatible PAP device**

Follow STEP 2 above. Run times will vary based on prescribed pressure, patient breathing pattern, mask seal, use of any additional features or attachments such as pressure relief, heated humidifier and heated tubing.

**NOTE:** For optimal run times while using your Pilot Flex with a compatible PAP device please disable and/or detach any humidifiers and heated tubing. Mask leaks or high PAP pressures can greatly decrease run times.

**Multiple Pilot Flex batteries can be linked together for additional run-time, please visit [www.medistrom.com](http://www.medistrom.com) for further instructions.**

## 1.7 PILOT FLEX SPECIFICATIONS

- Capacity: 98Wh
- Cell Type: LG/Samsung Industrial grade genuine rechargeable lithium ion cells
- Rated Input: 12V-24V 60W; USB-C: 5V-20V 45W Max
- Rated Output: DC 12V/24V 60W; USB-C: 5V-20V 45W (30W if main output is in use); USB-A 5V-12V 18W Max
- Life Span: Over 500 full charge/discharge cycles
- Case: Aluminum
- Dimensions: 8cm x 17cm x 2cm, Weight 1.2lbs
- Power Gauge: 4 stage Charge Level Indicator Lights
- Recharge time: 2-3 hours (7 to 9 hours via the ResMed AirMini AC Adapter and 4-6 hours via a 45W USB-C charger)

## SECTION 2 - CARE INSTRUCTIONS FOR PILOT FLEX

**CAUTION! Fully charge your Pilot Flex before storage and at least once EVERY 6 MONTHS thereafter. Failure to do this will be considered negligence and will void the warranty.**

CAUTION! Do not expose the battery to liquids of any kind. Keep out of direct sunlight.

If your unit becomes soiled, disconnect it from the power source and the PAP device and wipe the exterior casing with a clean, dry cloth.

## SECTION 3 - TRAVELLING WITH PILOT FLEX BATTERY BACKUP POWER SUPPLY

The Pilot Flex Battery Backup Power Supplies each have a 98Wh capacity. Many airlines will allow batteries under 100Wh to be taken into the cabin for travel. Please always consult with your particular airline and flight to confirm that your Pilot Flex can be taken with you in your carry-on prior to travelling.

If you plan on charging your Pilot Flex with the car charger, please make sure to test the car charger before travelling. If you plan on travelling and you have not used your Pilot Flex in the battery configuration, we strongly suggest that you test run the unit in the battery configuration prior to travelling in order to get an idea of what run-time you could expect.

For optimal run times please disable any additional features such as the heated hose, the humidifier, EPR as well as properly adjust your mask to minimize air leakage.

If you have any other inquiries or would like to download the Medistrom Travel Letter please visit [www.medistrom.com](http://www.medistrom.com) or [www.choiceonemedical.com](http://www.choiceonemedical.com). For further technical support please email [support@medistrom.com](mailto:support@medistrom.com).

## **SECTION 4 - STORAGE AND DISPOSAL OF YOUR PILOT FLEX BATTERY BACKUP POWER SUPPLY**

Store the Pilot Flex in a cool, dark, well-ventilated place with normal temperatures of 0 °C / 32 °F to 35 °C / 95 °F.

**CAUTION!** Keep out of direct sunlight! The disposal of this unit should be carried out in accordance with applicable laws and regulations reflecting the purchaser's geographic location.

For more information on where to recycle this product, please call 1-877-723-1297 or visit: [www.call2recycle.org](http://www.call2recycle.org)

## **SECTION 5 - WARNINGS AND SAFETY PRECAUTIONS**

- Please read this product manual carefully in its entirety before putting the equipment into operation for the first time. Observe all safety instructions.
- Do not expose the Pilot Flex to any hard, physical impact.
- Do not expose the Pilot Flex to water, rain or moisture of any kind.
- Do not expose the Pilot Flex to an open flame or extreme heat, including direct sunlight. Exposure to any of these may result in damage to the lithium-ion cells and may cause an explosion.
- Do not disassemble or attempt to repair the Pilot Flex.
- Ensure the product is thoroughly inspected for any physical damage. If any damage is found, the equipment should not be used.
- Any improper use or operation of the device, i.e., that is not consistent with the instructions, will result in forfeiture of all claims under the manufacturer's warranty.
- Prevent the risk of electric short circuit by using only the appropriate AC adapters and DC output cables outlined in this manual.
- If the battery pack becomes damaged and is leaking, do not touch the battery leakage directly: electrolyte leakage can cause skin irritation. In case the electrolyte comes into contact with the eyes, rinse with water as soon as possible. Do not rub eyes. Seek medical attention as soon as possible.

- Do not operate the device if the battery housing or AC adapter you are using is damaged. Immediately disconnect the damaged device from the power by pulling the plug and disposing of the unit appropriately.
- Do not expose the equipment to extreme temperatures, aggressive chemical vapors, moisture, shocks, vibration, or strong electromagnetic fields.
- Do not allow liquids to enter the device.
- Keep out of reach of children.

## **SECTION 6 - WARRANTY INFORMATION**

Please contact your CPAP battery dealer for any warranty related issues or you can contact Medistrom technical support by emailing [support@medistrom.com](mailto:support@medistrom.com).

Medistrom guarantees that Pilot Flex Battery Backup Power Supply shall remain free from manufacturer defects for twelve months from the date of purchase.

This warranty is not transferrable and will only be available to the original purchaser. This warranty does not cover physical damage resulting from improper use. Medistrom reserves the right to choose whether to repair or replace the product.

**CAUTION! Fully charge your Medistrom Pilot Flex Battery Backup Power Supply before storage and EVERY 6 MONTHS. Failing to do so will void the warranty.**

Medistrom reserves the right to update the warranty policy without notice. For the most up to date warranty information please visit [www.medistrom.com](http://www.medistrom.com) or contact support by emailing [support@medistrom.com](mailto:support@medistrom.com).

## SECTION 7 - TROUBLESHOOTING GUIDE

### NORMAL OPERATION

#### Green Charge Level Indicator Lights

4 solid green lights	Greater than 90% capacity remaining
3 solid green lights	65% - 90% capacity remaining
2 solid green lights	40% - 65% capacity remaining
1 solid green light	10% - 40% capacity remaining
Green lights flashing consecutively	Battery is charging

In the scenario where the Pilot Flex is experiencing overcurrent, overvoltage, or overtemperature, the battery will indicate an error with a flashing pattern of the Voltage indicator light. To stop the alarm state please disconnect all cables and press on the multifunction button.

Errors will be displayed as blinking lights: 500ms (short) and 1000ms (long).

Please refer to the table below for details.

### ERROR CODES

Error	Possible Cause	Solution
LEDs alternate flashing (Over Discharge State)	Battery has entered over-discharge protection to prevent damage to the cells.	Contact Medistrom technical support at 1-888-407-4791 (North America), 1-300-146-358 (Australia) or email support@medistrom.com
LED flashing rapidly (500ms)	Incorrect AC adapter	Pilot Flex: use the AC adapter provided with your Respirationics DreamStation or Respirationics System One S60 or other compatible 12V power adapters (see section 1.2).  Or you can use the AC adapter provided with your ResMed AirSense 10/11, AirMini or the Medistrom AC adapter (see section 1.2).
	Power source is damaged	Power strip or outlet wiring may be damaged. Plug AC adapter directly into wall outlet or try using the battery with a different wall outlet.
LED flashing long (1s) pattern	The environmental temperature is too high for the battery to operate safely	Wait for 10-30 minutes before using the battery and ensure that the battery is placed where there is no excess heat.  Do not rest the battery on top of radiators or electronic devices that may emit heat. Do not attempt to use the battery in temperatures exceeding 40° C / 104° F
LED flashing in a pattern of short (500ms) short (500ms) long (1s)	PAP device connected to battery is incompatible	Ensure that the PAP device you are using is compatible with the Pilot Flex (see section 1.2)
	DC output cable is damaged	Ensure that the cable connecting the Pilot Flex to your compatible PAP device is not damaged

LED flashing in pattern of three short (500ms) flashes	PAP device connected to battery is incompatible	Ensure that the PAP device you are using is compatible with the Pilot Flex (see section 1.2)
LED periodically flashing (no pattern)	Maximum battery power output	<b>No action required, the battery is operating correctly.</b>  Depending on your prescribed pressure, enabled PAP device features and breathing rate the battery may reach maximum power output which is indicated by the periodically flashing blue LED
LED flashing every 0.5 seconds	Battery capacity is below 5%	<b>Charge the battery as soon as possible.</b>  The battery power level is critically low. Not charging the battery for a prolonged period will cause it to go into Over Discharge State.
Battery Not Providing Power (PAP Device or USB)	Battery is in Stand-by Mode	Press the Multi Function Button to wake the battery from Stand-by Mode  Disconnect all devices and cables. Plug your DC output cable into the DC Out port on the battery and press the Multi Function Button  For USB Devices: Disconnect all devices and cables. Plug your DC output cable into the DC Out port on the battery then plug in your USB device into the USB Out port and press the Multi Function Button.

If errors persist after taking the recommended action(s) contact Medistrom technical support toll-free at 1-888-407-4791 (North America), 1-300-146-358 (Australia) or via email at support@medistrom.com.

Do not disassemble the device or attempt to service the device yourself.

## SECTION 8 - SYMBOLES

The following symbols may appear on the product or packaging.

Legal Manufacturer

Federal communications commission

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